

Directions for Use / **Debubblizer**

22320 Debubblizer, 8oz spray

22321 Debubblizer, 32oz refill

Kerr Debubblizer is a water-based surface tension reducer that allows close adaption of investments to all surfaces of the wax pattern.

Direct Application

- Apply by spraying, brushing or dipping the wax pattern or impression.
- Cover all surfaces of the wax pattern prior to investing or the impression with a smooth, even coat.
- Shake or blow away excess solution.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.