

Directions for Use / **Kleen Lube**

013-331 Kleen Lube, 15ml bottle

013-333 Kleen Dip Reservoir, 15 ml bottle, dropper

Kleen Lube is a specially developed, micro film lubricant created for the dipping Wax Coping technique that prevents adhesion of dipping wax to stone, epoxy, or metal dies and models.

Direct Application to the Die

Remove any excess liquid from the brush and apply directly a thin coat of Kleen Lube to the die.

Kleen Dip Reservoir

The Kleen dip reservoir has been designed for the efficient application of Kleen Lube to prepare dies.

Filling and Priming the Kleen Dip Reservoir

Use only Kleen Lube or Slick Lube in the Kleen Dip Reservoir. Other lubricants may contain chemicals that could deteriorate the Kleen Dip components.

- Unscrew and remove the cap from the Kleen Dip Reservoir.
- Fill the reservoir through the center of the sponge with 8 to 10ml Kleen Lube. (Do not overfill)
- The sponge absorbs the Kleen Lube and will deposit a uniform, micro thin coating of Kleen Lube on the surface of the die.

Note: Before using the Kleen Dip reservoir, it has to stand upright for at least 10 minutes so that the sponge becomes evenly saturated.

- Plunge a die into the slits in the Kleen Dip sponge. Insert past the ditch to lubricate the entire die. Rotate the die in the sponge 1-2 times and remove it. (See picture 1) The insertion and rotation will evenly coat the die with Kleen Lube and absorb any excess lubricant.

Note: When not using the Kleen Dip, keep the cap closed to minimize contamination and evaporation of the lubricant.



Picture 1

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.