

Directions for Use / **Laboratory Solitine**

22324 Laboratory Solitine, 8oz spray

Kerr Laboratory Solitine™ is a wax solvent and cleaner for use with most waxes and acrylics or porcelain teeth.

Direct Application

- After final wax-up, apply Laboratory Solitine to all exposed tooth surfaces with a cotton pledget and thoroughly work into interproximal and around gingival margins.
- While teeth are still wet with dissolved wax, hold wax-up under cold tap water and vigorously wash with a piece of cotton.
- Dry wax-up and polish teeth with a soft cloth or tissue.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.