

## Directions for Use / **Microfilm**

22322 Microfilm, 8oz spray

22323 Microfilm, 32oz refill

Kerr Microfilm™ is a non-oily water soluble die lubricant that allows an accurate wax pattern to be removed from the die without distortion.

### Direct Application

- On stone dies, use Microfilm liberally by brushing, by dipping, or by spraying the model surface.
- Allow a minute or two for Microfilm to penetrate then blow away excess.
- A second application will be necessary if the wax pattern is not waxed immediately.
- On metal dies, use Microfilm sparingly, blowing or wiping away excess solution before waxing.

Note: Do not soak stone dies in Microfilm for extended periods.

### LIMITED WARRANTY

#### LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.