

Directions for Use / **Softissue Moulage**

25794 Softissue Moulage, Intro Kit

26205 Softissue Moulage Separating Medium, 15ml bottle

28629 Softissue Moulage Refill Cartridges, 2-pack, 50ml each

With the advent of implants, the accuracy of the fit around the implant and the establishment of proper emergence angles, a soft resilient tissue model must be prepared to translate accuracy to the stone cast. Softissue Moulage can also be used on all applications of soft tissue references on models for aesthetics and for reference on crown and bridge techniques.

Extruder Gun Preparation

- Insert a cartridge into an extruder gun and lock it in place.
- Depress plunger until it touches the cartridge.
- Remove cap and extrude ¼ inch material. Wipe off with tissue.
- Twist on and lock a mixing tip to the cartridge.
- Cut off about 1/8 inch of the syringe tip and snap the tip into the end of the mixing tip.

Note: Do not extrude the material in the mixing tip until ready to use.

Impression / Model Preparation

- Follow the recommended aseptic and sterilization of the impression. Rinse and dry.
- Paint on a thin coat of separating agent around the implant area and all areas where you need the Softissue Moulage to separate.
- Place analogs in the impression and secure.
- Extrude the Softissue Moulage around the analogs and surrounding areas where you need the Softissue Moulage coverage. Build up the Softissue Moulage to approximately 3mm and allow setting for approximately 4-1/2 minutes.

Note: Do not cover the analogs with the Softissue Moulage as the analogs have to be locked in the stone.

- With a sharp bard parker blade, trim excess material away from the sulcus of the adjacent teeth, if needed.
- Pour a stone cast in the impression and allow setting per manufacturer's instructions.
- Separate and trim as usual. Softissue Moulage model is complete.

Note: If the Softissue Moulage material remains in the impression, carefully pry it loose and seat it back on the stone cast. You need to have it removable to check the seat of the abutments into the analog.

Clean Up and Preparation for Next Use

- When the material is set (approximately 4-1/2 minutes) it will peel off instruments and hands with ease.
- Leave the old mixing tip in place and remove only when starting a new procedure. When you remove the old mixing tip, extrude about ¼ inch of material and wipe it off with a tissue. Insert new mixing tip in place. The unit is now ready to use.

Note: Do not extrude material into the mixer until you are ready to extrude the Softissue Moulage material into the impression.

Caution

- Store at ambient temperature.
- Avoid prolonged exposure to excessive heat.
- Avoid direct contact with clothing of uncured materials.
- This material is not to be used for relining partials or dentures or in the mouth.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.