

Directions for Use / **Super-Sep**

22325 Super-Sep, 8oz spray
22326 Super-Sep, 32oz refill

Kerr Super-Sep™ is an alcohol-based separating fluid for stone or plaster that leaves no film and dries quickly.

Direct Application

- Apply by spraying or brushing the model. Cover all surfaces of the model with a smooth, even coat.
- Pour stone or plaster against it immediately. In some cases, two coats may be needed.
- Shake or blow away excess solution.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.