

## Directions for Use / **Vacufilm**

22327 Vacufilm, 8oz spray

22328 Vacufilm, 32oz refill

Kerr Vacufilm is an alcohol-based surface tension reducer that allows close adaptation of investment to all surfaces of the wax pattern. Vacufilm is designed to provide optimum results with vacuum casting technique.

### Direct Application

- Apply by spraying, brushing or dipping the wax pattern.
- Cover all surfaces of the wax pattern prior to investing with a smooth, even coat.
- Shake or blow away excess solution.

### LIMITED WARRANTY

#### LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr's product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.