DIRECTIONS FOR **Kerr** HYDROCK/RAPID STONE

ENGLISH

28599A
REVA698

Proportions: Approximately 30 ml of water to 100 grams of Hydrock/Rapid Stone by weight. This should be a thick creamy mix, somewhat thicker than an ordinary plaster mix. Avoid a putty-like consistency.

Mixing: Add the Hydrock/Rapid Stone to the water and spatulate vigorously with a stiff spatula for one minute. Pour immediately. Light vibration or jarring will easily flow the material to place. Separate when set.

Kerr Hydrock/Rapid Stone may be used for all denture work and light soldering. Treat it like any stone or plaster and excellent results will be obtained.

CAUTION

Inhalation of dry powder can be hazardous to your health. Use with adequate ventilation to meet TLV requirement. Use NIOSH-approved respirator for nuisance dust. Gypsums are subject to deterioration when exposed to atmosphere, particularly if the humidity is high. Reclose plastic liner immediately after each use.

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PHYSICAL PROPERTIES

Recommended water-powder ratio: 30 ml of water to 100 grams of powder.

Setting Time (Vicat):	10 - 12 mins.
Setting Expansion:	0 - 0.18%
Crush Strength (1 hr):	3500 psi
Shelf Life: Refer to package marking. Do not use after expiration date.	

Store at ambient conditions

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists or laboratories in using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist or laboratories of testing Kerr's products to determine their suitability for the intended uses and procedures. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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