DIRECTIONS FOR
Kerr
SNOW WHITE PLASTER No. 2

ENGLISH

28592A
REVA698

No. 2 (Rapid Setting 3 - 5 Minutes)

FOR IMPRESSIONS - By Volume - Approx. two (2) parts
Water to three (3) parts Plaster.
(94 ml of Water to 100 g of Powder)

DIRECTIONS FOR MIXING PLASTER No. 2

Mix equal parts of water and material in a clean bowl;
spatulate for 15 seconds. To this thin paint-like wash add
the remaining part of material gradually and spatulate
thoroughly for 15 more seconds with a rapid whipping
stroke. This will ensure equal crystallization throughout the
entire mass. Eliminate air bubbles by rotating bowl or
patting or jarring on sides.

WHEN TO POUR

Continue to spatulate rapidly until thick creamy consistency
is obtained; then pour.

NOTE:

The above proportions are approximate. As these materials
absorb moisture from the air, slight variations in the
proportions may be necessary, depending upon the
atmospheric conditions. It is important to use distilled water
when the natural water of a locality contains a high
percentage of soluble salts, etc.

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CAUTION

Inhalation of dry powder can be hazardous to your health.
Use with adequate ventilation to meet TLV requirement. Use
NIOSH-approved respirator for nuisance dust. Gypsums are
subject to deterioration when exposed to atmosphere,
particularly if the humidity is high.

Shelf Life: Refer to package marking. Do not use
after expiration date.

Store at ambient conditions.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is
designed to assist dentists in using Kerr's product. Such
advice does not expand Kerr's limited warranty or relieve the
dentist of testing Kerr's products to determine their suitability
for the intended uses and procedures. The dentist assumes
all risk and liability for damages arising out of the improper
use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's
liability is limited, at Kerr's option, to replacement of the
defective product or part thereof, or reimbursement of the
actual cost of the defective product. In order to take
advantage of this limited warranty, the defective product must
be returned to Kerr. In no event shall Kerr be liable for any
indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE
NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED,
INCLUDING WARRANTIES WITH RESPECT TO
DESCRIPTION, QUALITY, OR FITNESS FOR A
PARTICULAR PURPOSE.

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Sybron Dental Specialties Japan, Inc.