DIRECTIONS FOR

SUPRASTONE
HIGH STRENGTH DIE STONE

ENGLISH

28602A
 REVA698

The recommended procedure of mixing Kerr Suprastone is to use a vacuum mixer. The recommended ratio is 19 ml of water to 100 grams of powder. The powder should be added to the water in the bowl of the mixer and incorporated by hand for 30 seconds. It should then be mechanically mixed under vacuum for an additional 30 seconds. If a multiple speed mixer is used, then the lowest speed should be selected. Light vibration will cause Kerr Suprastone to flow readily into the impression. Because of its extreme accuracy and hardness, Suprastone High Strength Die Stone is recommended for master models and dies for all cast work, denture models, study models and general Laboratory use.

CAUTION

Inhalation of dry powder can be hazardous to your health. Use with adequate ventilation to meet TLV requirement. Use NIOSH-approved respirator for nuisance dust. Gypsums are subject to deterioration when exposed to atmosphere, particularly if the humidity is high.

PHYSICAL PROPERTIES

Recommended water-powder ratio: 19 ml of water to 100 grams of powder.

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Setting Expansion: 0.15 - 0.25%
 Crush Strength (1 hr): 7500 psi
 Shelf Life: Refer to package marking. Do not use after expiration date.
 Store at ambient conditions.

**LIMITED WARRANTY
 LIMITATION OF KERR'S LIABILITY**

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. Such advice does not expand Kerr's limited warranty or relieve the dentist of testing Kerr's products to determine their suitability for the intended uses and procedures. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr U.K. Ltd.
Sybron Dental Specialties Japan, Inc.