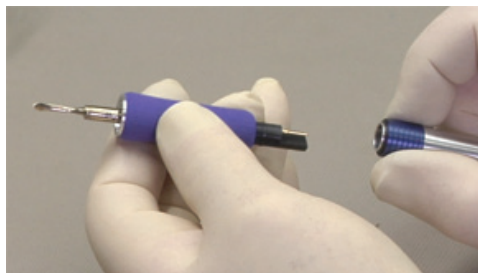
## Ultra-Spatula / Tip Directions for Use

### Removing a Tip

Retract the spring loaded Quick Connect Collar (Blue) towards the cord end of the Ultra-Spatula handle. The movement of the collar will be about 0.3 cm. Gently remove the Heating Tip in the opposite direction.

**CAUTION** Handle a hot tip with care to prevent burns.

### Replacing a Tip

Retract the Quick Connect Collar (Blue) and insert the Heating Tip about half way. Gently rotate the Heating Tip until you locate the connection (two flat sides) that will allow the Heating Tip to be inserted the rest of the way.

Use the Quick Heat feature (under Menu) once you have changed to a new tip, the Quick Heat feature will bring the cold Tip to a working temperature quickly.

## Ultra-Waxer Heating Tips

1 2 3 4 5 6 7 8 9 10

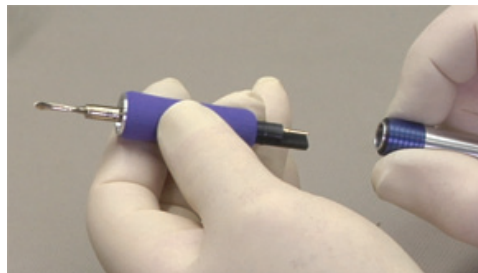
- 35167 Small PKT, Blue
- 35168 Large PKT, Teal
- 35169 Beavertail, Purple
- 35170 Small #7, Grey
- 35171 Large #7, Green
- 35172 Needle, Red
- 35173 ½ Hollenback, Black
- 35174 Small Denture, Pink
- 35175 Large Denture, Orange
- 35176 Denture Spoon, Yellow
- 35177 Ultra-Spatula Cord/Handle

**HOT SURFACES** - Do not touch external and internal exposed parts of the unit.

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## LIMITED WARRANTY

#### LIMITATION OF KERR'S LIABILITY

Kerr’s technical advice, whether verbal or in writing, is designed to assist dentists and laboratories in using Kerr’s product. The dentist or laboratory assumes all risk and liability for damages arising out of the improper use of Kerr’s product. In the event of a defect in material or workmanship, Kerr’s liability is limited, at Kerr’s option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liablefor any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

Manufactured By
**Dental Laboratory Products**
Kerr Corporation
Orange, CA 92867 USA
Fax: 714.516.7649
kerrlab.com

European Union Representative
**Kerr Italia S.r.l.**
Via Passanti, 332
I-84018 Scafati
(SA) Italy
Tel. +39-081-850-8311
kerrdental.eu

Kerr Distribution Facilities
**KerrHawe SA**
Via Strecce 4
CH-6934 Bioggio
Switzerland
Tel: +41-91-610-0505
Fax: +41-91-610-05-14
kerrdental.eu

**Kerr Australia Pty. Ltd.**
112-118 Talavera Road, Unit 10
North Ryde 2113
New South Wales, Australia
Tel: +61-2-8870-3000
Fax: +61-29-870-7600
kerrdental.com

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**Sybron Dental Specialties Japan, Inc.**