
Galetti Articulator®

Plasterless Fixed Plane Articulator

- English** *Instruction for use*
- Italiano** *Istruzioni per l'uso*
- Deutsch** *Anwendung*
- Français** *Mode d'emploi*
- Español** *Utilización*
- Dansk** *Anvendelse*

KerrHawe SA
Via Strecce 4
6934 Bioggio/Switzerland
www.KerrHawe.com
00800 41 05 05 05

Kerr USA
Dental Laboratory Products
Orange, CA 92867
Fax 714.516.7649
www.kerrlab.com

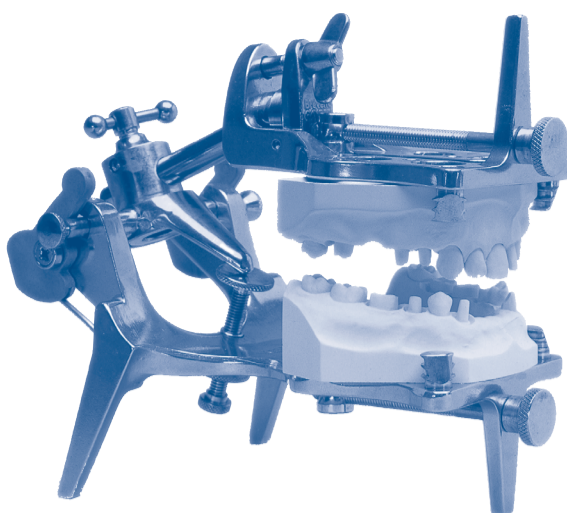Item Number Europe 61332 - Item Number USA 32685

The Galetti Articulator 18th

The worlds most recognised Plaster - Free Articulator by Kerr.

Solidly constructed from Aluminium and Brass. Hard wearing and corrosion resistant. Easy to use within the laboratory or at chair side. Highly accurate. Time saving.

Any type of dental model can be placed onto the Galetti articulator without the application of mounting plaster. This enables precise and controlled adjustment of the occlusion to exacting tolerance. Both upper and lower jaws may be separated allowing for greater access and stability.

Laboratory and Chair side Use

1. Check functional bite registration.
2. Assess Inter Cuspal Position (Centric Occlusion).
3. Adjust occlusal interferences of dental prostheses.
4. Assist with planning occlusal schemes.

The Galetti articulator enables the user to check occlusal and articulated balance for several types of dental prostheses. Constant assessment of mandibular motion may be verified by adjustment of the articulator.

Partial Dentures: Free articulation will be determined by the residual natural teeth. This may be achieved by loosening the anterior spherical joint.

Complete Dentures: Balanced occlusion for Anterior, Posterior and Lateral excursion, may be determined by allowing the models to freely articulate at the Hinge Axis. Work the prostheses in its desired direction for selective adjustment. This will eliminate any cuspal interference.

Orthodontics: Models may be mounted to assess ongoing correction and reduction.

Crown and Bridge Restorative: Functional and trial wax up prior to completion can be assessed depending upon the type of occlusal scheme required.

How to use:

- Cast and trim models so that they will fit onto the upper and lower jaws.
- Loosen the posterior and anterior ball joint.
- Apply the upper and lower models into their respected jaw.
- Tighten the anterior traction screws to firmly fix and locate each model.
- Adjust the models into a correct occlusal relationship.
- Tighten the Posterior joint.
- Tighten the anterior joint.

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