

Kerr Corporation Take 1 Advanced VPS Impression Material


Take 1 Advanced combines strength, elasticity, and dimensional stability to create impressions in any environment.

According to Kerr Corporation, Take 1 Advanced is the first VPS impression material to not only focus on just a single feature but also combine various characteristics to meet many individual clinical needs. The manufacturer says that each feature of Take 1 Advanced rates as good as or better than any other VPS material on the market, and that the collective performance of these features exceeds that of competing products.

Detail and Precision of Impressions

In research conducted by Kerr, it was found that the chemistry of Take 1 Advanced creates a very wettable surface. Because the surface is so wettable, it breaks surface tension and displaces fluids in the oral cavity to capture detailed impressions with no voids or gaps.

When the 6 dentists who participated in this evaluation were asked to rate Take 1 Advanced on the detail and precision of the impressions they created, 3 rated

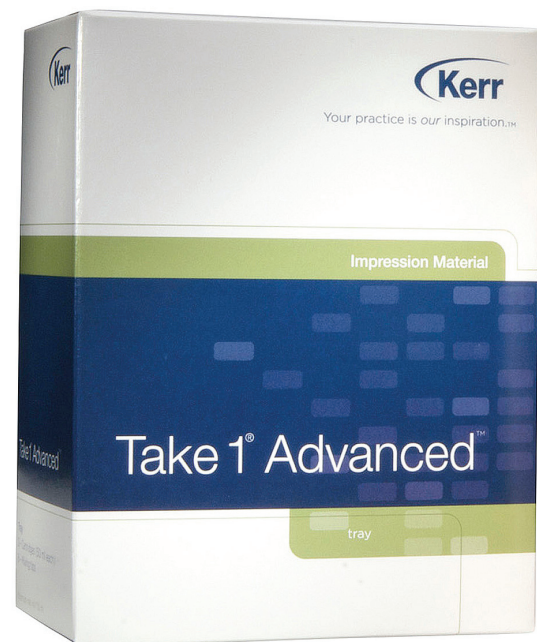

it as excellent, 2 rated it as very good, and 1 rated it as good. One evaluator described details captured in the preparation as “quite good.” Another said it was a “very readable material with good body and rigidity.” This evaluator went on to say that the “preparation details were easy to evaluate.” A third evaluator said,