



Your practice is *our* inspiration.™



Manufactured for:
Kerr Corporation
1717 West Collins Ave
Orange, CA 92867
800.KERR.123
kerrdental.com

MZ 4141 a
05/09

Hemostasyl™

Hemostatic Dressing with Mechanical Action

Instructions For Use

- Hemostatic Dressing with Mechanical Action

1 - PRESENTATION

• HEMOSTASYL is presented in 2g syringe and is used with single-use angled application cannulas (Syringe and application cannulas are also sold separately).

2 - PROPERTIES

HEMOSTASYL is an original paste to be used as a haemostatic dressing with a mechanical action. The main characteristics of HEMOSTASYL are as follows: • thixotropic properties • good adhesion to the gingival mucosa, without compression • presence of aluminium chloride which reinforces its mechanical haemostatic effect • placed directly in the mouth with the single-use angled cannula • easily located due to its colour • easily removed by gentle water spray without resumption of bleeding • painless method when HEMOSTASYL is used on a healthy periodontium • contact time: 2 minutes

3 - COMPOSITION

• Aluminium chloride. • Excipients.

4 - INDICATIONS

HEMOSTASYL Paste achieves its haemostatic effect in the gingival sulcus by physical and mechanical means during dental procedures, such as dental impressions, scaling, seating of temporary and permanent restorations, and placement of a rubber dam.

5 - PRECAUTIONS FOR USE

• Cannulas are disposable. DISPOSE OF the used cannulas after use. • Do not store cannulas in contact with syringes. Store new cannulas in their original packaging, away from syringes (risk of corrosion). • The use of plastic syringe covers is recommended. • HEMOSTASYL is not indicated in the case of heavy bleeding. • Do not use in a site where the product may not be completely removed. • Ensure complete removal of HEMOSTASYL. • In case of contact with the eyes, rinse with plenty of water. • Do not make swallowed. • Keep out of the reach of children.

6 - CONTRA-INDICATIONS

• Extraction sockets. • Root canal surgery. • Known allergy to aluminium chloride.

7 - INSTRUCTIONS FOR USE

• **Preparation:** Open the syringe. Screw the single-use angled application cannula onto the tip of the syringe.

• **Application:** The paste is applied directly onto the bleeding gum to completely cover the bleeding site, without exerting any pressure on the gum with the cannula. Leave in place for 2 minutes.

• **Removal:** Remove HEMOSTASYL by gentle water spray and simultaneous aspiration. Verify that HEMOSTASYL is completely removed from the gum.

8 - DISINFECTING THE OUTSIDE OF THE SYRINGE

Once the single-use cannula has been removed and discarded, the outside of the syringe can be cleaned and decontaminated using (aldehyde-free) disinfectant wipes in accordance with the manufacturer's directions for use.

9 - STORAGE

• After use, imperatively remove and discard the used cannula. Tightly close the syringe. • Do not use past the expiry date indicated on the outer packaging. • Store at temperature < 25 °C.

10 - DISPOSAL

• Do not discard into sewers or streams. • Do not discard with household waste.

Professional dental product.

June 2009



Kerr Corporation

1717 W. Collins Ave., Orange, CA 92867
800-KERR-123 KerrDental.com

Fabricated by Produits Dentaires Pierre Rolland For Kerr
Hemostasyl is a registered trademark and
entire property of Produits Dentaires Pierre Rolland
FRANCE



Kerr Corporation

1717 W. Collins Ave., Orange, CA 92867
800-KERR-123 KerrDental.com

Fabricated by Produits Dentaires Pierre Rolland For Kerr
Hemostasyl is a registered trademark and
entire property of Produits Dentaires Pierre Rolland
FRANCE